

AMPRIA®


Post-Operative Rigid Dressing Instructions



1. With the AMPRIA® opened, the foam cup below the residual limb and patient in a supine position, place AMPRIA beneath the waist and affected limb. Align the end of the limb **beyond** the bottom edge of the device by 1”.




4. Secure waist section high around your waist, tightly. You should feel snug contact of the bottom cup against the end of your limb. There should be no void between the cup and your limb.

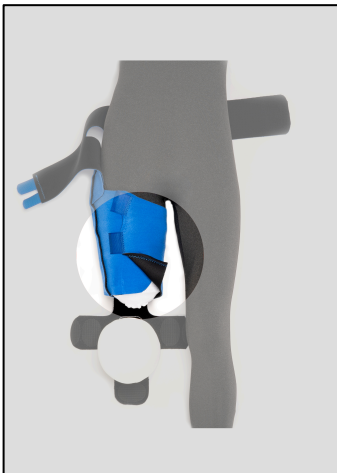

2. Wrap thigh portion around leg, overlapping material, and secure “marked” tab first. Beginning at top, secure tabs snugly along midline of leg except for bottom tab. Sleeve should feel comfortably snug as tabs are secured.



5. FINISHED! This is the appearance when correctly applied.

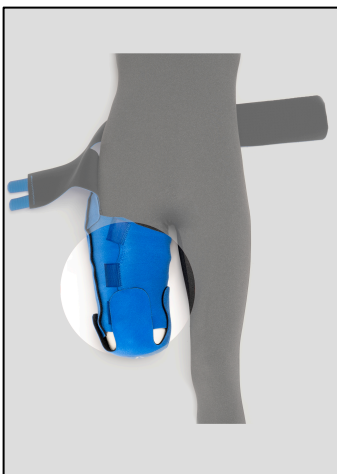

3. Rotate bottom foam cup inside wrap to contact the limb. Fasten last tab in front over edge of cup, then secure the 3 large tabs snugly, holding cup against the bottom of the limb

QUESTIONS? Contact us at info@ampria.net or (207) 831-0471.