



AMPRIA®


Post-Operative Rigid Dressing Instructions

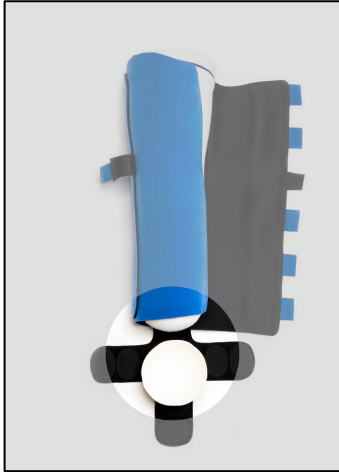


1. Place the AMPRIA® beneath the leg with the tabbed edge laid out to the side and the foam cup laid out below the leg. Align the end of your limb **beyond the bottom edge of the device.**

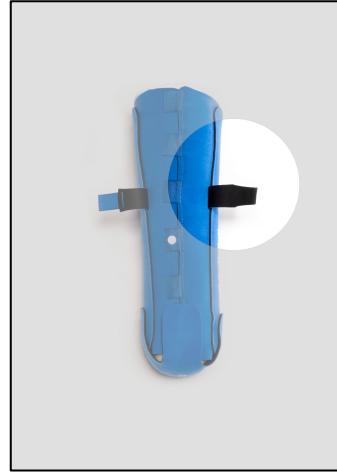

2. Wrap the portion with tabs around the leg and secure the marked tab first. **Beginning at top,** secure tabs snugly along the midline of the leg except for bottom tab. The sleeve should feel comfortably snug as tabs are secured.

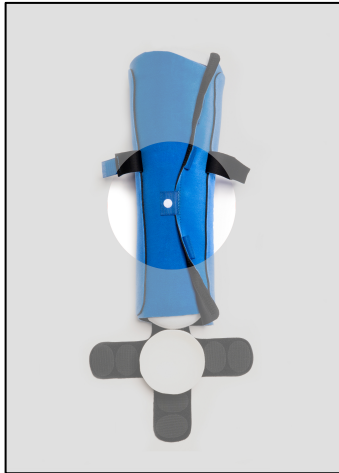

3. Rotate bottom foam cup inside the wrap to contact with the limb, fasten the last tab in front over the edge of the cup and then secure the 3 large tabs snugly holding the cup against the bottom of the limb

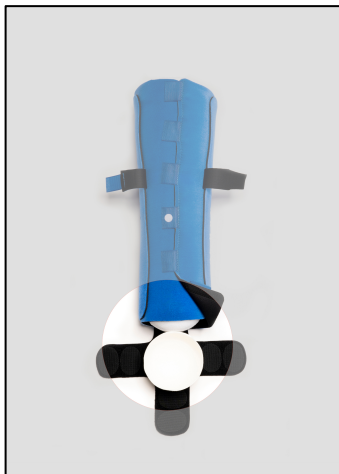

4. Secure the thigh strap tightly 1" - 2" above the knee cap to aid suspension.



5. FINISHED! This is the appearance when correctly applied.

QUESTIONS? Contact us at info@ampria.net or (207) 880-1150.